



Safety

The belt system has been comfort tested by the Swedish National Road and Transport Research Institute (VTI). The tests show that crash test dummies weighing 22, 34 and 75 kg remain fixed to the seat at full brake from 90 km/h.

The belt system

- Can be fitted in all vehicles with seats where there is an opening between seat and backrest.
- Can be fitted in both front and back seats (but not in fixed back seats where special adaptation is required).
- Can be used in child safety car seats (forward-facing) that have an opening between the seat and the back.
- Can be used in conjunction with a belt cushion.
- Can be fitted in all wheelchairs with a fixed back and an opening between the seat and the backrest.



Shoulder Belt with Chest Buckle

Supplementary articles with longer delivery time

- Shoulder Belt in sizes S, M and L.
- Chest Buckle for Shoulder Belt (S, M, L)
- T-Belt with buckles at the sides (S, M, L)
- Basic lock in 3 different lengths for T-Belt
- Combination Belt, without the Spinal Belt, buckles at the sides or at front (S, M, L)

Please contact Autoadapt for information on the supplementary articles.
support@autoadapt.se
Phone: +46 302 25440

CARE BELT®

Careva
system

POSTURE BELTS



For further
information

www.autoadapt.com

**auto
ADAPT**
for your independence

Autoadapt AB, Åkerivägen 7,
S-443 61 Stenkullen, Sweden
Tel: +46 (0)302 25400 Fax: +46 (0)302 25407
E mail: contact@autoadapt.se www.autoadapt.com

Autoadapt UK Ltd, Unit 1, Windsor Industrial Estate,
Rupert Street, Aston, B7 4 Birmingham, UK
Phone: 0121 3335170 Fax: 0121 3596194
E mail: contact@autoadapt.co.uk www.autoadapt.co.uk

Developed as a complement
to provide all disabled persons
with the comfort and safety
during car travel.

**auto
ADAPT**
for your independence

CARE BELT[®] system

- posture belts for a better sitting position




The Spinal Belt is strapped around the backrest and has 5 grooves, in which the posture belts are mounted.



It is simple and quick to buckle and unbuckle the belts.




The Careva belt is CE- marked (which indicates product compliance with the EU medical aids directive).

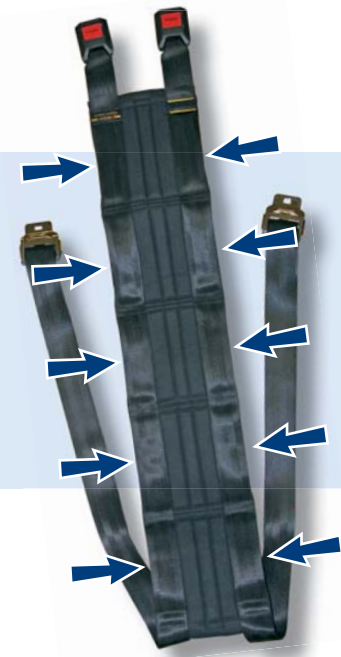


The Spinal Belt

Combination Belt, kit with Spinal Belt included

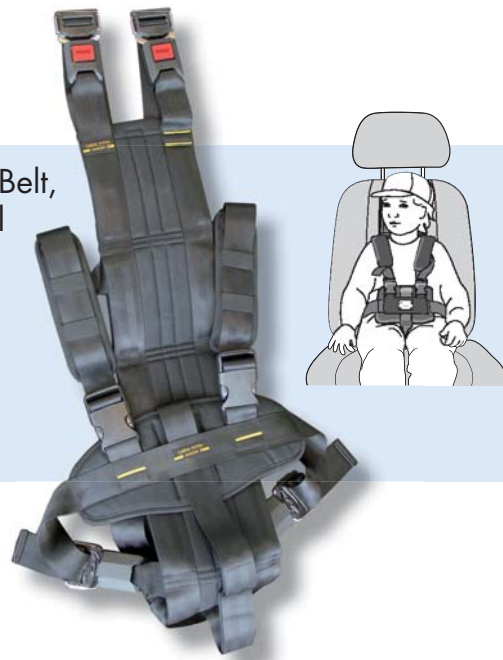

Thoracic Belt

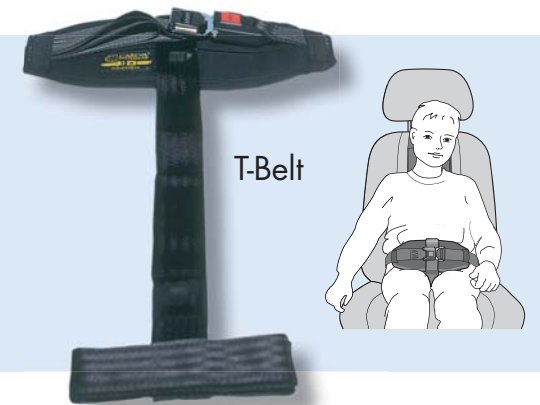

T-Belt

The mobility-impaired often require extra support or restraint to ensure a positive sitting position in cars, wheelchairs, child car safety seats, etc.

The Spinal Belt forms the basic structure of the belt system and is strapped around the backrest of the seat. After that a suitable posture belt is mounted in the grooves of the Spinal Belt, to achieve an upright body position. To provide the best possible fit, the belts are ordered by size; Small, Medium or Large and in some models by length.

Note! The vehicle seat belt must always be used.

The Careva belt System is complementary to the vehicle seat-belt and helps support the body to an upright position.

The Careva Belt system assures the driver that the passenger is sitting in an optimal position. An upright body position is important in order to optimize the function of the vehicle seat-belt, in case of a crash.

The belts are available in black and can be machine washed in 40 degrees, provided that the metal buckles are removed to spare the washing machine.

The Careva Belt can be used together with a seatbelt cushion and /or a car safety seat (provided that it has an opening between backrest and seat).

The system is also a great help for Taxi and Mobility services and can quickly and easily be mounted and remounted in the different vehicles.

To prevent active individuals from unbuckling their belts during the ride, a basic lock of 150 cm is available for buckling behind the backrest.



A long Basic Lock of 150 cm is available for buckling behind the backrest.



The Careva Thoracic Belt



The Shoulder Belt (article for order, with a longer delivery time)



The Combination Belt, together with the Carony Kids transportation wheelchair.

The Careva belts are also very well fitted for use in a wheelchair with a fixed backrest.

CARE BELT system

Careva Demo Kit

The Careva demo kit includes one of each belt in medium size, plus a demo plate. With the Careva demo kit you can show your customers all the different combination options and how to solve different seating problems!

You order the demo kit directly from us with article number:
101412.
The price of the demo kit is
£ 431.



**We offer you a
Careva demo kit!**

 **auto
ADAPT**
for your independence

INVENTIONS FOR QUALITY OF LIFE


These belts are included in the Careva Demo Kit:



Chest buckle for
shoulder harness



Buckle

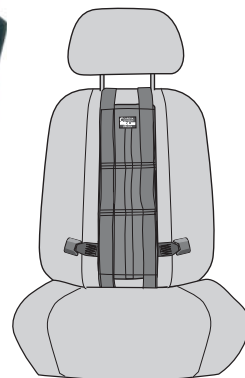


Shoulder Harness




Thoracic Belt



Combi Harness

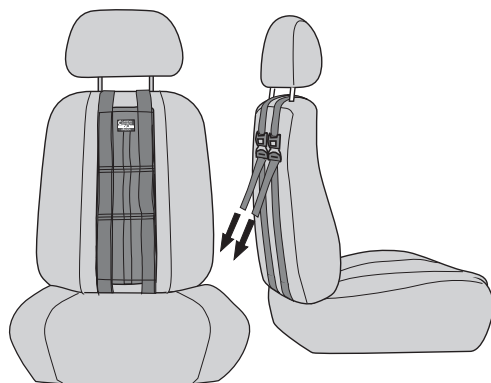

Spinal Belt with two buckles
and five channels.

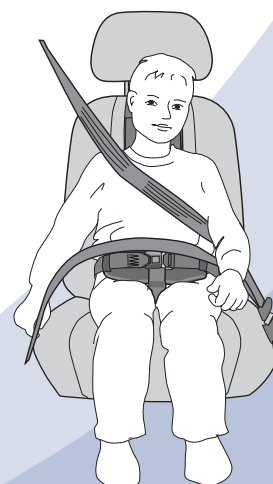

Pelvic T-belt



Autoadapt UK Ltd.
Unit 4, Windsor Industrial Estate
Rupert Street, Aston
B7 4PR Birmingham,
U.K.
Phone: +0121333 5170
E-mail: contact@autoadapt.co.uk
www.autoadapt.co.uk

